


ADJUSTMENT OF THE IRRIGATION FLOW IS ESSENTIAL

In order to obtain good drop-by-drop irrigation suitable for periodontal treatments, and a spray that does not create aerosol, the adjustment of irrigation must be adapted to each tip.

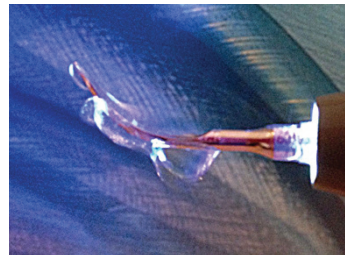

- 1 Set the irrigation flow rate to 0 and the power to 3 on the ultrasonic generator.
- 2 Hold the handpiece with the tip pointing upwards.
- 3 Adjust the spray by progressively increasing the flow rate so that the irrigation sprays the tip point with drop-by-drop
- 4 Set the machine to the required power.
- 5 Start working with aspiration close to the tip

TIP SETTINGS FOR THE ULTIMATE PERFORMANCE



Tips \ Pure NEWTRON Units	POWER	IRRIGATION
---------------------------	-------	------------

PROPHYLAXIS

1 / 2 / 3 / 1S	14	
10P	14	
10X / 10Z	12	
EX1 / EX2 / EX3 / EXL / EXR	12	

PERIODONTICS

H1 / H2L / H2R / H3 / H4L / H4R	2	
P2L / P2R	3	
TK1-1S	2	
TK1-1L / TK2-1L / TK2-1R	2	

Tips \ Pure NEWTRON Units	POWER	IRRIGATION
---------------------------	-------	------------

ENDODONTICS

ET20D / ET25L / ET40 / ET40D	7	
IRR20-21 / IRR20-25 / IRR25-21 / IRR25-25	9	
K10 / K15 / K25 / K30	6	
AS3D / AS6D	7	
AS9D	6	
ASLD / ASRD	7	
P14D / S12-70D	7	
P15LD / P15RD	7	
SO4	7	

IMPLANT CARE

PH1 / PH2L / PH2R	2	
IP1	3	
IP2L / IP2R / IP3L / IP3R	5	

ENDODONTICS


CAP1	10	
CAP2 / CAP3	10	
ET18D	10	
ET20 / ET25 / ET25S / ETBD	7	

PROSTHESIS & AESTHETICS

PM1 / PMS1	15	
PM2 / PMS2	12	
PM3 / PMS3	10	
PM4	14	
PMV1 / PMV2 / PMV3	15	
PMV4 / PMV5 / PMV6	10	
5AE / ETPR	20	
C20	11	

CHECK YOUR TIP WEAR BEFORE USE

Place the tip over the corresponding diagram to see if it is suitable to use.
Please discard if they do not cross the red line

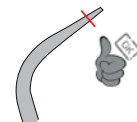

PERIO TIPS



H1



H2R



H2L



H3




H4R




H4L


SCALER TIPS

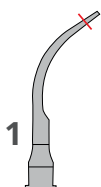


1

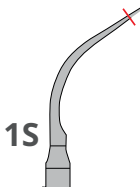


1S

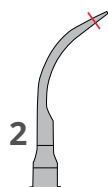


2

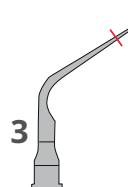


3

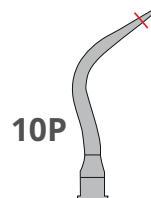

10P

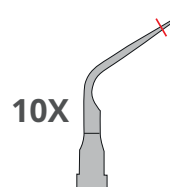

10X

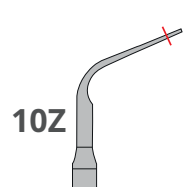

10Z