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Fig. 3

DESCRIPTION

Control Unit for use with the DEFINITIVE®LED brushless electric micromotor. The Unit allows activation of the micromotor by means of the foot pedal of the dental Chair.

The micromotor is meant to drive a dental handpiece with gear ratio 1:1 or 1:5. It is anyway possible to connect handpieces with different gear ratios.

The system is intended for use in general dentistry and restorative dentistry by dentists and dental professionals in a dental office.

The Unit:

- must be used only by trained and qualified personnel.
- does not produce electromagnetic interference.
- is not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- is not designed to be placed on wet surfaces or to come in contact with liquids.

This medical device meets the requirements of the European Directive 93/42 EEC (Class IIa).

CONTENTS

Control unit, DEFINITIVE®LED brushless electric micromotor, detachable silicone hose, external AC/DC power supply adaptor, power cord.

UNIT INLETS AND OUTLETS (FIG. 1)

- a) 4-way connector for the Turbine hose, air and water inlet.
- b) Output connector for the silicone hose for supply of the electric micromotor.
- c) Power socket for the external power supply adaptor.
- d) Power switch.

INSTALLATION

Position the Unit on a table, on a trolley or on the dental Chair. Protect the Unit from direct sunlight and dust. Keep the original packaging for storage and shipment.

With reference to Fig.2, carry out the following steps:

- a) Connect the Turbine hose coming from the dental Chair by screwing the corresponding nut; then tighten it clockwise.
- b) Connect the output silicone hose by screwing the corresponding nut; then tighten it clockwise.
- c) Connect the power cord to the external power supply adaptor and insert its output plug into the power socket.
- d) Turn power switch on. Activation of the Unit is shown by the LED **f** on the front panel (Fig.3).

SPEED ADJUSTMENT

The speed of the micromotor can be adjusted by means of the front knob **e** (Fig.3).

Adjustment range is from 1000 (1x1000) to 40000 (40x1000) RPM for a handpiece with 1:1 transmission ratio and from 5000 (1x5000) to 200000 (40x5000) RPM for a handpiece with 1:5 transmission ratio.

ROTATION DIRECTION

The rotation direction can be toggled by pressing the corresponding **g** button on the front panel (Fig.3). The LED is off if the preset direction is clockwise (Forward); the LED is on if the preset direction is counter-clockwise (Reverse).

FLOW ADJUSTMENT

The flow of spray air and water can only be adjusted on the dental Chair to which the Turbine hose is connected.

ACTIVATION

Start of the electric micromotor is carried out by pressing the foot pedal of the dental Chair to which the Turbine hose is connected. If the foot pedal is pressed before the Unit is powered up then the micromotor cannot be started; in this case release and press the foot pedal again.

USE OF THE MICROMOTOR

For connection, use and maintenance of the DEFINITIVE®LED micromotor, refer to the corresponding operating instructions.

CLEANING AND DISINFECTION

Cleaning of control Unit: rub the surfaces with a clean cloth soaked in a ordinary detergent product or with a disinfecting disposable wipe recommended for surgical instruments.

Cleaning of the detachable silicone hose: use a disinfectant based on alcohols like ethanol, methanol or a product containing glutaraldehyde up to 2%.

Do not use disinfectants containing benzalkonium chloride or acetone.

Do not immerse in a disinfectant solution or in a ultrasonic bath.

GUARANTEE

Either the Unit or the micromotor carry a 24-month guarantee against all defects of material or faulty construction. Guarantee validity is subject to specific conditions.

SERVICE

Never disassemble the device. In case of malfunction or failure of the integrity of the enclosure, and for any overhaul and repair work, please contact your usual supplier or directly to TKD.

REFUSE DISPOSAL

This device needs to be recycled. Electrical and electronic equipment may contain dangerous substances which constitute health and environmental hazards. The user must return the equipment to its dealer or establish direct contact with an approved body able to process and derive value from this type of equipment (European Directive 2002/96/EC).

TECHNICAL DATA

Power supply: 100 ÷ 240 V~ (alternating current), 50/60 Hz, 135 W.

Motor rotation speed: 1000 ÷ 40000 RPM

Degree of protection against electrical shock: Class I, type B applied part according to CEI EN 60601-1.

Degree of protection against ingress of water: ordinary IPX0 equipment.

Max. pressure of spray air and water: 4 bar (58 psi).

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